

Breathe Easier— Wherever Life Takes You

If you live with COPD, bronchiectasis, cystic fibrosis, spinal-cord injury, or a neuromuscular condition, thick mucus can make every breath harder. BiWaze® Clear is designed to help: its dual-blower system gently opens your airways and loosens trapped mucus in a typical 10-minute session.*

By moving secretions toward the larger airways, BiWaze Clear may make it easier to cough or suction them out, so you can breathe more comfortably.

With a lightweight design and a built-in battery, you can take your therapy from the living room to the backyard—or even on the road—without missing a session.



Included with every BiWaze® Clear system:

- Control unit with an internal battery -up to 10 standard therapies on a full charge*
- Convenient carrying case
- Dual Lumen Breathing Circuit™ which includes a coaxial filter, coaxial breathing tube, Aerogen® nebulizer, handset, and patient interface (mouthpiece, face mask or trach adapter)
- Power cables for the control unit and Aerogen® Solo nebulizer
- Instructions for use manual and quick reference guide




* Nominal = a 10 minute therapy.

abm.
RESPIRATORY CARE

BiWaze® and ABM Respiratory Care® are registered trademarks of ABMRC, LLC.
©2023 ABMRC, LLC. ALL RIGHTS RESERVED.
PRTN-1585539264-871 Rev 2.0 JUL-2025

For further information about this product or a service, visit our webpage:
www.abmrc.com or call customer support: 1-877-ABM-RC01 (877-226-7201)

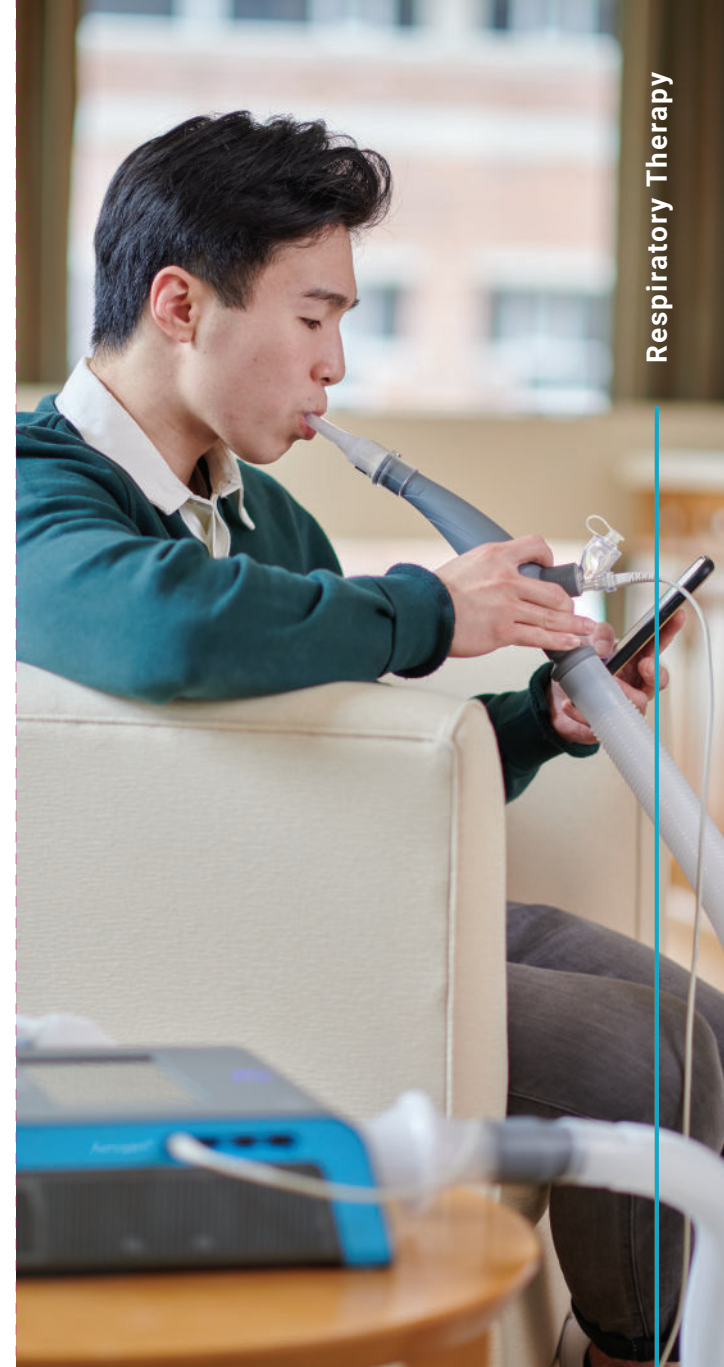

BiWaze® Clear
SYSTEM

One system, three therapies

The BiWaze® Clear System combines three therapies:

1. Positive Expiratory Pressure (PEP):

The system delivers a programmed positive pressure you exhale against to open and expand your airways.



2. High Frequency Oscillation:

The system gives pulses of positive pressure to thin mucus and move it higher up inside the lungs where it can be coughed or suctioned out.



3. Nebulizer:

The system uses a nebulizer to deliver medication into the lungs, either during the PEP and oscillation treatments, or by itself.



Internal or External Oscillation — which May Be Right for You?

Why might your doctor prescribe airway-oscillation therapy?

- Thick mucus that's hard to cough up
- Frequent lung infections
- Limited mobility after surgery or illness

How do the two options compare?

External oscillation (HFCWO) — vest therapy

What it is: A vest or wrap gently vibrates your chest for **about 30 minutes** to loosen and move mucus.

Best for you if: you prefer not to use a face mask, mouthpiece, or trach adapter.

Internal oscillation (OLE) — BiWaze® Clear

What it is: Gentle air pulses delivered through a face mask, mouthpiece, or trach adapter clear mucus **in just 10 minutes** while also providing lung expansion and nebulization.

Best for you if: a vest is uncomfortable (due to surgical incisions, burns, ports, osteoporosis, or chest sensitivity) or if you want to combine multiple airway-clearance therapies to shorten treatment time.

Work with your clinician to decide whether **internal** or **external** oscillation therapy is the right fit for your care.

